



## HISTOSMATE

An innovative, safe, rapid solution for the efficient cleaning of wax contaminated racks, molds and forceps.

✓ **FULLY AUTOMATIC OPERATION**

- 1) Load the rack - The cavity holds non-toxic isoparaffin at a constant 50°C.
- 2) Push START - The heated isoparaffin is constantly stirred to assure rapid and efficient elimination of wax traces.
- 3) Unload the clean rack - **The entire process takes 7 minutes.**

✓ **NO XYLENE**



PATHOS RACK AFTER PROCESSING

**AUTOMATIC,  
RAPID RACK  
CLEANING**



AFTER 7' CLEANING WITH HISTOMATE



## Need more wax impregnation time?

### Dual use: the HistosMATE as a stand alone, automatic wax impregnation module

- 1) Simply load rack in the cavity which holds paraffin wax at a user's selected temperature.
- 2) Set desired impregnation time (up to 24 hours).
- 3) The optimum combination of powerful magnetic stirring and homogeneous temperature distribution assures a complete wax impregnation of specimens.

HistosMATE  
universal stainless  
steel holder suitable  
for all types of racks



#### OPTION

HistosMATE steel basket to automatically remove wax traces from metal molds and forceps.



Before



Seven minutes later



**MILESTONE**

UNI EN ISO 9001: 2000 CERTIFIED

MILESTONE S.r.l.  
Via Fatebenefratelli 1/5  
24010 Sorisole (BG) Italy  
Tel. +39-035-4128264  
Tel. +39-035-572735  
Fax +39-035-575498  
e-mail: [medical@milestonesrl.com](mailto:medical@milestonesrl.com)  
[customersupport@milestonemedsrl.com](mailto:customersupport@milestonemedsrl.com)  
[www.milestonemedsrl.com](http://www.milestonemedsrl.com)

MILESTONE MEDICAL  
25 Controls Drive  
Shelton, CT 06484 - USA  
Tel: (203) 925-4240  
Toll-free: (866) 995-5300  
Fax: (203) 925-4241  
e-mail: [info@milestonemed.com](mailto:info@milestonemed.com)  
[www.milestonemed.com](http://www.milestonemed.com)